

# **Professional Cubicle Curtains**

1-800-258-8817 • custsvc@ModoMed.com www.ModoMed.com

# **CUBICLE CURTAIN CARE AND SPECIFICATIONS**

**FABRIC: Sing** 

#### **FEATURES:**

- Greenguard Gold Certified
- LBC Red List Free
- PFAS Free

#### SPECIFICATIONS:

- Fabric Content: 100% FR Polyester
- Finish: None
- Additional Finishes Available: Alta Privacy Curtain, Nanotex with Antimicrobial
- Backing: NoneWidth: 72"
- Repeat: 72" V, 18½" H
- Railroaded Repeat: 18½" V, 72" H

# **Flammability Tests Passed:**

- AS/NZS 1530.3
- ASTM E 84 Unadhered
- CAN ULC S109
- GB/T 17591
- NFPA 701

# **Durability Tests Passed:**

Lightfastness: AATCC 16E, 60+ hours



### **CARE INSTRUCTIONS:**

- Using mild soap or detergent, machine wash on synthetic setting with a high water level and mild agitation. Temperature should not exceed 160F. Extract gently. Wash with like materials.
- If necessary, machine wash with a diluted bleach solution. This textile may also be spot cleaned with a diluted bleach solution as long as the bleach solution is rinsed thoroughly after application. If bleach is used without adequate rinsing, degradation to the textile will occur.
- Air, cabinet, or tumble dry at a low temperature. Do not exceed 3-5 minutes on a synthetic cycle at a maximum of 140F. Hang or fold immediately after drying.
- Hand Iron Iron at low heat setting (275F) with a damp press cloth or steam iron with a dry press cloth.
- Grid Head Press Press for short intervals with minimum steam. Do not lock head.
- Flat Bed Press Press dampened drapery using cloth covering.
- Avoid prolonged contact with heat.

<sup>\*</sup>ModoMed relies solely upon the flame retardant tests from the mills and makes no claims, directly or implied, regarding flame retardant properties of any fabrics.